

Natural aesthetics —
Accurately reproduced



ReGum Dental light-cure composite resin

About ReGum


ReGum is an organic dental restorative material. Its fluid-state properties enable effortless infiltration into minute areas, faithfully replicating the color and morphology of natural gingival tissues. Designed for morphological contouring and aesthetic recreation of the gingival portion in dentures.

☑ Stability

Excellent shape retention;
minimal stringing; not easy to slumping.

☑ Ease of Use

Simple application procedure;
user-friendly.

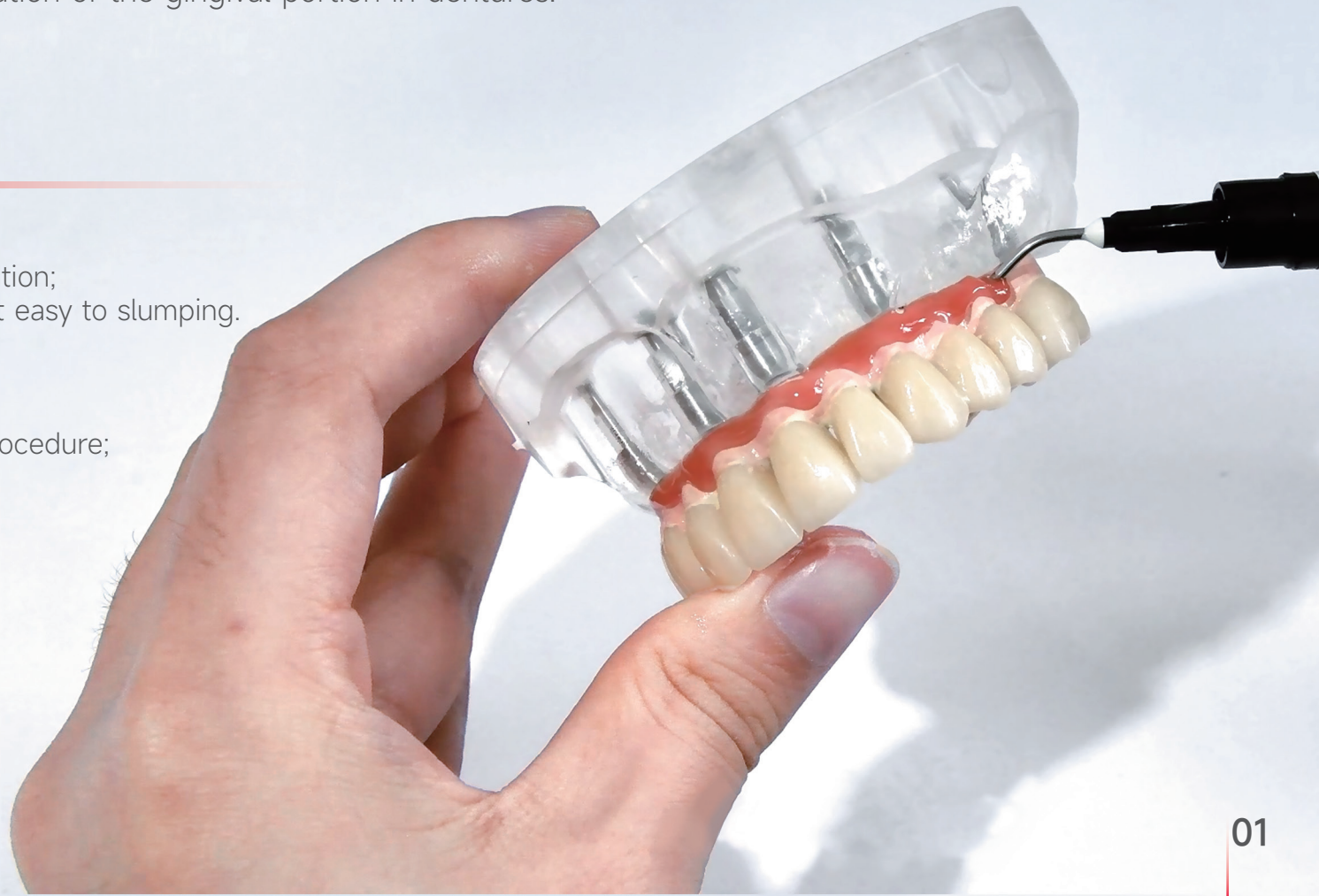

Highlight



Superior Material Properties

German-sourced high-transmittance glass powder ensures high strength, lasting color stability, and fade-resistant aging performance.



Rapid Curing

Light-cures in just 40 seconds, significantly boosting technician productivity.



Comprehensive & Natural Shades

A variety of shades can be customized to simulate the color and texture of natural gums.

(G1, G2, G3, G4, M4, T1/T2, OP-W, OP-P)




Exceptional Opacity

OP opaque ceramic effectively masks metal substrates, achieving enhanced color realism.



Six steps to perfect gingival aesthetics

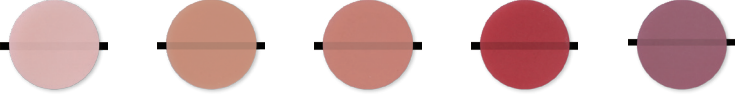
Opaque	Translucent	Concentrate	Specification
 OP-W OP-P	 T1 T2	 G1-Light G2-Rosa G3-Pink G4-Red M4-Lila	3g/pc

1 Color masking




Apply the OP-P and OP-W to mask the color.
Mix different proportions to create different shades of gum base color.

2 Base Coating

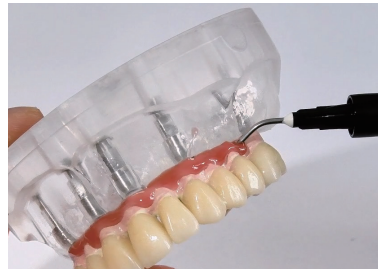

Apply **G3-Pink** with graduated thickness.
Light-cure for 40 seconds.

3 Embolus Sculpting

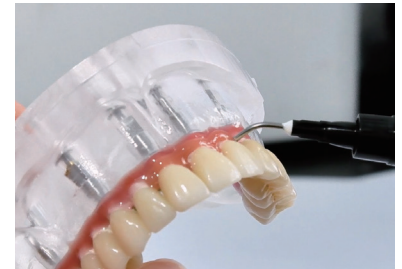

Concentrate **G2-Rosa** resin to build three-dimensional contouring.

4 Capillary Simulation



Trace localized **G4-Red** streaks to mimic mucosal vasculature.

5 Gradient Blending



Blend **G2-Rosa** with **T1** or
Apply thin **T1** layer for natural highlight transition

6 Detail Refinement



Accentuate frenulum lines and free gingival margins with **G1-Light**.

Living Quality: Brought to Denture Restoration



✓ Must-Have

● G2-Rosa
Enhance Anatomic Contouring

● G3-Pink
Establish Base Hue

⊖ Recommended

● T1

● G1-Light

● G4-Red

ABOUT US

DEPRAG is a global dental technology enterprise specializing in digital dental materials, smart devices, and cloud-based services, with comprehensive coverage across restorative, orthodontic, and implant dentistry. Through material science innovation and digital ecosystem development, the company is driving the transformation of dental care from traditional experience-driven practices to a digitally-driven model.

Zhengzhou

Headquarters
Dental Materials and Software R&D Center
Marketing Center
Dental Lab

Shenzhen

Dental Equipment R&D, Manufacturing,
and Training Center
Product Display Center

Anyang

Dental Materials
Manufacturing Center



- 25+ Excellent dealer
- 1000+ Employees
- 90+ Countries and Regions



◀ Factory Video

DEPRAG (Zhengzhou) Dental Technology Co., Ltd.

Web: depragdental.com

Email: info@depragdental.com

Add: No.14, No.8 Floor, Building 7, No.289 West Third Ring Road, High-tech Industrial Development Zone, Zhengzhou City, Henan Province, China.

